



QUICK REFERENCE GUIDE

Record the MAX & MIN temperatures twice daily - early morning & late afternoon. This will show what has happened during the day & overnight:

Reading & Recording the MINIMUM & MAXIMUM temperatures

- Before pressing any buttons, note the currently display temperature on your Temperature Record Sheet.
- Press and hold the '▼' to show the MIN temperature, and note this on your Temperature Record Sheet.
- Press and hold the '▲' to show the MAX temperature, and note this on your Temperature Record Sheet.

Resetting the MINIMUM & MAXIMUM temperatures

- Press and hold the 'X' button until you hear a bleep.
- MIN & MAX temperatures have both now been reset to the current temperature.

Alarm Display

If the temperature falls below +2c or rises above +8c for more than 15 minutes, the alarm will be activated. 'Hi' or 'Lo' will flash in the display and the buzzer will sound. Press the 'X' button to silence the buzzer.

Automatic Defrost

During the automatic defrost period 'def' will displayed. Normal storage temperatures are maintained during this defrosting period.