



QUICK REFERENCE GUIDE

Record the MAX & MIN temperatures twice daily - early morning & late afternoon. This will show what has happened during the day & overnight:

Recording the MINIMUM temperature:

- Press & hold the ◀-◀◀ (MIN) button
- Record the displayed temperature
- Then push SET button whilst pressing the ◀-◀◀ (MIN) button
- The MIN temperature has now been reset.

Recording the MAXIMUM temperature:

- Press & hold the ▶-▶▶ (MAX) button
- Record the displayed temperature
- Then push SET button whilst pressing the ▶-▶▶ (MAX) button
- The MAX temperature has now been reset.

Switching ON or OFF

You can switch ON or OFF the refrigerator by pressing and holding for a few seconds the AUX button. 'oFF' will be displayed when the appliance is off. *(This feature may not be available on all models)*

Alarm Display

If the temperature falls below +2c or rises above +8c for more than 15 minutes, the alarm will be activated. 'Hi' or 'Lo' will flash in the display and the buzzer will sound. Press & hold either ◀-◀◀ or ▶-▶▶ buttons to silence the buzzer.

Automatic Defrost

During the automatic defrost period 'def' will be displayed. Normal storage temperatures are maintained during the period.